- (DE) Gebrauchsanweisung
- (FI) Käyttöohjeet
- (GB) Instructions for use
- (GR) Οδηγίες Χρήσης
- (FR) Mode d'emploi
- (CZ) Návod k použití
- (ES) Instrucciones de uso
- (HU) Használati utasítás
- (IT) Istruzioni per l'uso
- (LV) Lietošanas instrukcija
- (PT) Instruções de uso
- (LT) Naudojimo instrukcija
- (NL) Gebruiksaanwijzing
- (PL) Instrukcja obsługi
- (SE) Bruksanvisning
- (DK) Brugervejledning
- (NO) Bruksanvisning

Manufacturer:
Kulzer GmbH
Leipzigser Straße 2
63450 Hanau (Germany)
Made in Romania
Importado e Distribuido no Brasil por
Kulzer South America Ltda.
CNPJ nº 48.708.010/0001-02
Rua Cenno Strighi, 27 - cj. 42
São Paulo - SP - CEP 05036-010
sac@kulzer-dental.com
Resp. Técnica: Dra. Regiane Marton - CRO 70.705
Nº ANVISA: vide embalagem

66056478/13

Technische Informationen

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Stand: 2016-07

Technical Information

Silicon based precision condensation curing material.

Instructions for use (GB)

Optosil® / Xantopren®	EN ISO 4823	Colours	Dosage**		Mixing-time	Working-time**	Time in mouth	
			BASE					
			Liquid	Paste				
Optosil® Comfort®	Typ 0	yellow	1 spoon (10,7 g/9,5 ml)	6-8 drops (0,14 g/0,13 ml)	6-8 gradations (0,32 g/0,29 ml)	30	1:00	4:00
Optosil® P Plus	Typ 0	yellow	1 spoon (14,4 g/9,5 ml)	4-6 drops (0,09 g/0,09 ml)	4-6 gradations (0,21 g/0,19 ml)	30	1:00	4:00
Xantopren® H green	Typ 1	green	6 gradations (11,0 g/8,8 ml)	12 drops (0,27 g/0,26 ml)	12 gradations (0,65 g/0,59 ml)	30	1:00	4:00
Xantopren® M mucosa	Typ 2	orange	12 gradations (9,0 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 gradations (0,65 g/0,59 ml)	30	1:30	4:00
Xantopren® L ⁴ blue	Typ 3	blue	12 gradations ¹ (8,6 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 gradations (0,65 g/0,59 ml)	30	1:30	4:00
Xantopren® VL ⁴ Plus	Typ 3	blue	12 gradations ¹ (8,5 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 gradations (0,65 g/0,59 ml)	30	1:30	4:00

*1 At room temperature of 23°C (73°F), 50% rel. humidity.
Higher temperatures shorten, lower temperatures prolong these times.
*2 1 gradation = 1,3 cm (applies to mixing pad available)
*3 Bottom ring on mixing dish is equivalent to 7,6 ml
*4 When using the dispenser: 1 stroke (4,5 g / 4,0 ml) + 6 drops (0,14 g / 0,13 ml)
The stated dosage should be observed.

Typ 0 = putty consistency
Typ 1 = heavy bodied consistency
Typ 2 = medium bodied consistency
Typ 3 = light bodied consistency

User Advice

The Optosil®/Xantopren® assortment is a complete system. Components can be individually combined for all impression techniques. For best results usage with other silicon materials and Activators is not recommended. Visually inspect for damage before first use. Damaged products must not be used. Close carefully after use.

Pouring the model:

Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days.

Recommended model material:

All commercially available modeling stones can be used.
Our suggestions:
Class III Gips, MOLDANO®/MOLDADUR®
Class IV Gips, MOLDASTONE®/MOLDASYN®

Electroplating:

Impressions may be copper or silver plated in the usual way. Impressions should generally be treated by immersion for 10 minutes* in 5,25% Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the product's manufacturer.
* Gelson Luis Adabo, DDS, PhD, Elaine Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD
Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621-4, 1999

Adhesion:

We always recommend the use of our Universal Adhesive. Do not store above 25°C (77°F).

Warning and precaution for Activator universal Plus:

Flammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. Wear protective gloves/protective clothing/eye protection/face protection (EN 374, category III, e.g. Dermatrill from KCL, Germany). Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Do not use in pregnant or nursing patients or in children under 6 years of age. For use by dental personnel only.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us and to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control, and therefore, entirely your own responsibility. Should, in spite of this, liability be incurred for any damages, it will be limited to the value of goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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